HARTMANN

Hydrosorb[®]

COMFORT

Hydrogelverband

Hydrogel dressing

Pansement hydrocellulaire en gel

Hydrogel wondverband

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Apósito de hidrogel

Penso de hidrogel

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Instructions for use

Hydrosorb/Hydrosorb comfort is a transparent hydrogel dressing consisting of an absorbent polyurethane gel covered with a gas-permeable polyurethane film which is impermeable to liquids and bacteria. The transparent, write-on documentation aid facilitates complete documentation of the wound-healing process.

Properties and mode of action

Hydrosorb/Hydrosorb comfort can release moisture and absorb secretions. This provides a moist wound environment which has a stimulating effect on wound healing. Hydrosorb/Hydrosorb comfort is transparent, does not stick to the wound and can be removed without leaving any residue. Because of these characteristics, wound inspection is possible at any time without a change of dressing. Irrigation is not necessary for assessing the wound during a change of dressing.

In addition Hydrosorb comfort has a hypoallergenic adhesive film border so that no other materials are needed to secure the dressing.

Indications

For all superficial wounds in the granulation and epithelialisation phase, and can also be used to soften and re-hydrate necrotic tissue.

Mode of application

Select a size of Hydrosorb/Hydrosorb comfort so that the dressing overlaps the margins of the wound by approx. 1 – 2 cm. Remove the blue protective foil on the back, place Hydrosorb/Hydrosorb comfort on the wound, press it down well and secure it. To fix securely to joints and rounded areas of the body, make cuts in the adhesive border of Hydrosorb comfort as necessary and overlap them to stick together.

To facilitate complete documentation of the wound-healing process, a transparent, write-on documentation aid is provided on the product. Note date and patient's name on the transparent documentation aid and trace the margins of the wound with an overhead marker. Then carefully pull the documentation foil off the wound dressing, remove the protective foil of the adhesive strip and stick the foil on the wound documentation form: see pictograms.

Hydrosorb: see pictograms 1a to 1h

Hydrosorb comfort: see pictograms 2a to 2h

As the exudation decreases, Hydrosorb/Hydrosorb comfort can remain in place for up to seven days. Absorption of secretions is apparent as clouding and/or blistering of the dressing. Hydrosorb/Hydrosorb comfort should be changed when the blister has approximately reached the dimensions of the wound area. To change the dressing, detach Hydrosorb comfort slightly at one edge and remove it carefully.

Side effects

No side effects have been observed.

Incompatibilities

Hydrosorb/Hydrosorb comfort may occasionally cause skin reactions in susceptible subjects.

Contraindications

Do not use on wounds which are clinically infected, likewise on 3rd degree burns.

Special precautions

Medical assessment of the condition of the wound and the causes of disturbed wound healing are necessary before treating wounds with impaired healing. Treatment with Hydrosorb/Hydrosorb comfort cannot replace treatment of the cause of the impaired healing.

General instructions

Store below 40 °C. Contents sterile unless peel package is damaged. Do not resterilise. Keep out of reach of children.



AU – PAUL HARTMANN Pty. Ltd. • Homebush NSW 2140

GB – PAUL HARTMANN Ltd. • Heywood/Lancashire OL10 2TT

ZA – HARTMANN-Vitamed (Pty) Ltd. • Rivonia Sandton 2128



1

2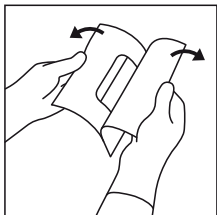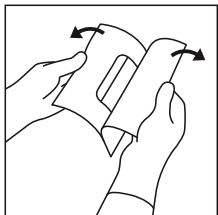



a

a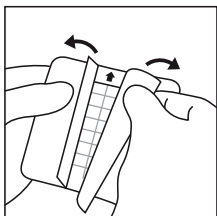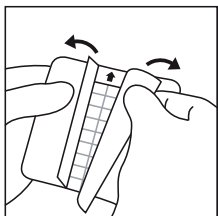



b

b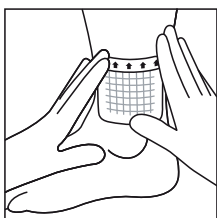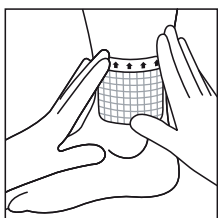



c

c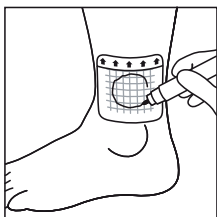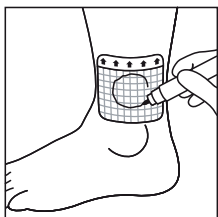



d

d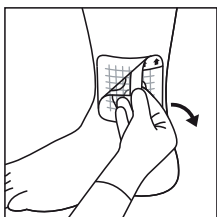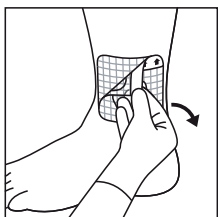



e

e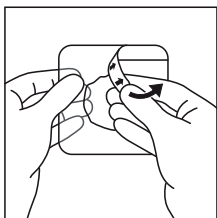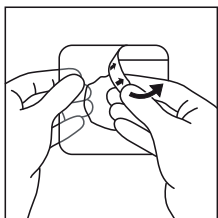



f

f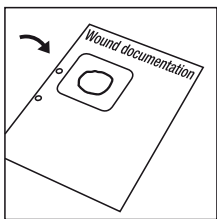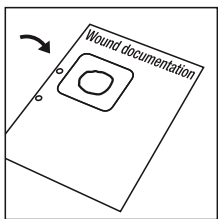

g

g

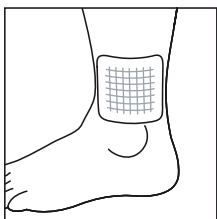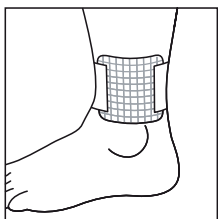h

h

HARTMANN

Hydrosorb®

Hydrogelverband
Hydrogel dressing
Pansement hydrocellulaire en gel
Hydrogel wondverband
Medicazione idrogel
Apósito de hidrogel
Penso de hidrogel
Επίθεμα υδροκυτταρικού ζελέ
Hydrogelový obvaz
Hydrogélóv ý obváz
Opatrunek hydrożelowy
Hidrogél sebfedő
Гидрогелевая повязка
Hydrogelförband
水凝膠敷料
ضمادة بالهلام المائي
하이드로겔 창상피복재

STERILE

R