HARTMANN

PermaFoam®

Schaumverband

Foam dressing

Pansement en mousse

Schuimverband

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Apósito de espuma

Penso de espuma

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Stand der Information: 2009-04

Instructions for use

PermaFoam is a polyurethane foam dressing with a gas-permeable, waterproof and germ-resistant outer layer.

PermaFoam comfort is an adhesive polyurethane foam dressing with a gas-permeable, waterproof and germ-resistant outer layer. PermaFoam comfort also has an adhesive border coated with a polyacrylate adhesive.

PermaFoam cavity is a polyurethane foam dressing with no outer layer, which is therefore effective on both sides.

Properties and mode of action

PermaFoam* absorbs exudate to create a moist wound environment that stimulates wound healing.

PermaFoam comfort, concave and sacral have adhesive borders so no additional materials are required to secure dressings.

PermaFoam is designed to prevent adhesion to the wound. Dressings can be removed painlessly and leave no residue.

Indications

PermaFoam dressings are particularly suitable for the treatment of medium to heavily exuding wounds.

PermaFoam cavity is suitable for deep, fissured wounds. As folded tamponades especially suitable for wound cavities.

PermaFoam tracheostomy is suitable for the care of tracheostomy cannulas and tubular applications.

Application instructions

Where necessary, clean the wound prior to applying the dressing. Select PermaFoam, PermaFoam comfort, sacral and concave according to the size of the wound so that the dressing overlaps the wound margins by approximately one to two centimetres. The larger the wound, the more the dressing should extend beyond the wound margins. Cut the dressing to size if necessary.

PermaFoam: Apply PermaFoam with the white side on the wound so that the beige outer layer is facing outwards, then secure the dressing.

PermaFoam comfort, sacral and concave: Remove the protective paper, apply the white side of the foam pad to the wound and press down the adhesive edges firmly.

To facilitate the removal of PermaFoam, the strength of the adhesive can be reduced by moistening it, e.g. with Ringer's solution. If the wound is uncomplicated, PermaFoam can remain on the wound for several days depending on the amount of secretion and the condition of the dressing.

A change of dressing is necessary if clinically indicated or if secretion appears around the edges.

PermaFoam sacral: see diagram 1

PermaFoam concave: see diagram 2

PermaFoam tracheostomy: see diagram 3

PermaFoam cavity: see diagram 4

Contraindications

Do not use PermaFoam in the event of symptomatic fungal infection.

Precautions

Prior to the treatment of wounds with impaired healing, a medical assessment of the wound condition and the cause of the healing disorder should be undertaken. Treatment with PermaFoam is no substitution for causal treatment of the wound healing disorder.

In the case of clinically infected wounds and in areas with exposed bones and tendons, PermaFoam should only be used under medical supervision.

In the event of deep second degree and third degree burns, PermaFoam may be used under medical supervision following excision of necrotic tissue or surgical treatment.

Do not use PermaFoam concurrently with oxidants such as oxychloride solutions or hydrogen peroxide. This can damage the foam.





Special instructions

Store in a dry place. Contents are sterile unless the peel packaging is damaged. Do not resterilise. Keep out of the reach of children.





* Where PermaFoam appears in the following text, this applies to all presentations; any exceptions to this are expressly mentioned.

Date of revision of the text: 2009-04





1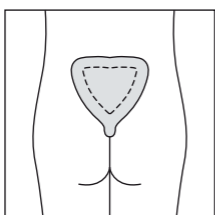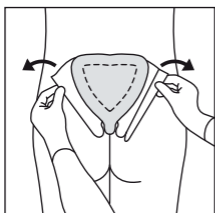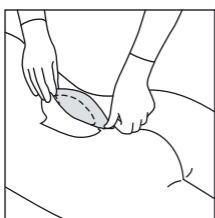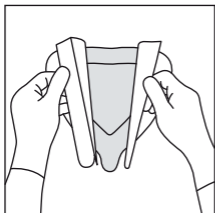

2

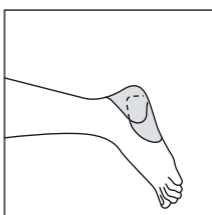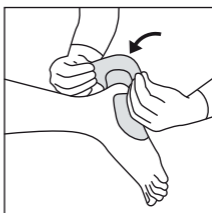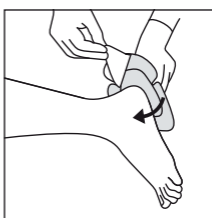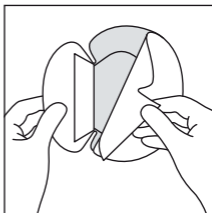3

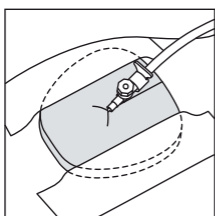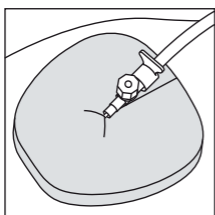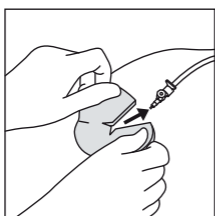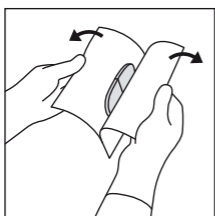4