HARTMANN

Sorbalgon/-T®

Calcium-Alginat Wundauflagen
Calcium alginate wound dressings
Pansements d'alginate de calcium
Calciumalginat-wondverband
Medicazioni in alginato de calcio
Apósitos de alginato de calcio
Pensos de alginato de cálcio
Επιθέματα αλγινικού ασβεστίου
Kalciumalginátové obvazy na rány
Kalciumalginátové obvazy na rány
Opatrunki z alginianu wapnia
Kalcium-alginát sebfedő anyag
Повязки из кальция-альгината
Kalcium-alginat-sårförband
Kalciumalginatbandager
Kalsium-alginaattisiteitä
Kalsium-alginat sårforbinding
藻鈣類傷口敷料
ضمادات كالسيوم ألجينات لتغطية الجروح

STERILE

R

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Instructions for use

Sorbalgon is a calcium alginate non-woven wound dressing (or ribbon). In contact with sodium salts – as found in wound exudate – the alginate fibres become gelatinous creating a moist wound environment. This in turn supports the natural healing process.

Properties & Mode of Action

The non-woven structure of the dressing combined with its physical and chemical properties result in a high absorbency – particularly important in the treatment of moderate to heavily exuding wounds. The material adapts remarkably to the contours of the wound preventing desiccation. For the treatment of cavity wounds, Sorbalgon T ribbons are available.

Indications

For all moderate to heavily exuding or bleeding wounds.

Application

Cleanse the wound in accordance with standard nursing practice. Apply the dressing to the wound or plug the wound with a Sorbalgon T ribbon and secure with a suitable secondary dressing. In case of dry wounds Sorbalgon can be moistened with TenderWet Solution or Ringer's solution. Sorbalgon may be left in-situ for extended periods (up to several days), depending on the level of wound secretion. The dressing should be changed – once it has fully turned into a gel – in accordance with nursing standards. Should any fibres remain in the wound, these may be easily re-moved by a wound irrigation solution* and are broken down naturally by the body.

Side effects

None have been observed or reported.

Contraindications

There are no known contraindications associated with the use of Sorbalgon on exuding wounds.

Special precautions

Medical assessment of the condition of the wound and the causes of the delayed wound healing are necessary before treating wounds with impaired healing. Treatment with Sorbalgon cannot replace treatment of the cause of the impaired healing.

General instructions

Store in cool and dry conditions. Contents sterile unless peel packs are damaged. Do not re-sterilize. Keep out of reach of children.

* Ringer's solution, Ringer's rinsing solution without any additives or TenderWet solution.

AU – PAUL HARTMANN Pty. Ltd. • Homebush NSW 2140

GB – PAUL HARTMANN Ltd. • Heywood/Lancashire OL10 2TT

ZA – HARTMANN-Vitamed (Pty) Ltd. • Rivonia Sandton 2128

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PAUL HARTMANN AG · 89522 Heidenheim, Germany
CH – IVF HARTMANN AG · 8212 Neuhausen