HARTMANN

# TenderWet® active

## CAVITY

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## **Directions for use**

TenderWet active cavity is a moist wound dressing whose central component is a superabsorbent polymer (SAP) activated by Ringer's Solution.

## **Properties and mode of action**

TenderWet active cavity releases Ringer's Solution into the wound. Interactive and continuous wound irrigation takes place since the dressing absorbs wound secretions at the same time.

## **Indications**

For wet treatment of wounds with impaired healing.

## **Mode of application**

TenderWet active cavity is applied or packed into the wound and secured with a bandage or, if necessary, with adhesive tape. TenderWet active cavity must be in contact with the base of the wound and should overlap the wound edges slightly. TenderWet active cavity must be changed after 12 hours at most or in combination with TenderWet 24 active after 24 hours. If the wound requires a large quantity of moisture, the dressing can dry out prematurely. In such cases TenderWet active cavity should be moistened further between dressing changes. Ringer's Solution can be used to irrigate underneath the dressing before it is removed to ensure a gentle dressing change; it can then be removed painlessly after a few minutes.

## **Side effects**

At the start of the treatment with TenderWet active cavity, breakdown of irreversibly damaged tissue can produce an increase in the size of the wound. This is a sign that wound healing has commenced. Reddening of the wound edges can occur during wet therapy; this is usually a sign that the blood circulation has been reactivated as the wound begins to heal. There can be a tendency to spontaneous bleeding from the wounds during therapy with TenderWet active cavity. This is usually an indication that the wound is starting to heal. However, the cause must be clarified in each case. TenderWet active cavity therapy as such does not cause any pain. However, open wounds are often very sensitive to mechanical stimuli. It often helps if the dressing is moistened again after a few hours (without changing the dressing); otherwise temporary administration of appropriate analgesics is indicated.

## **Contraindications**

There are no known contraindications to date.

## **Special precautions**

TenderWet active cavity must not be cut to size or damaged mechanically in other ways. There is only limited experience of combining TenderWet active cavity with topical agents. Wet therapy can lead to disintegration of the plantar horny layer. Ringer's Solution does not lead to maceration of the intact epidermis. Wet therapy with Ringer's Solution does not lead to maceration of vital cells. However, the plantar horny layer can swell, which is visible as a whitish layer. It is therefore advisable to apply a greasy ointment or a water-in-oil emulsion to the wound edges. It should be ensured that no residues remain on the skin.

**CAUTION:** Medical assessment of the condition of the wound and the causes of slow wound healing is necessary before treating wounds with impaired healing. Treatment with TenderWet active cavity cannot replace treatment of the cause of impaired healing.

## **General instructions**

Store below 25° C. Contents sterile unless peel pack is damaged. Do not re-sterilise. Keep out of reach of children.

**PAUL HARTMANN AG · 89522 Heidenheim, Germany**

- AT – PAUL HARTMANN Ges.m.b.H. · 2355 Wiener Neudorf**
- AU – PAUL HARTMANN PTY LTD · 34 McEvoy St · Waterloo · NSW 2017**
- BE – N.V. PAUL HARTMANN S.A. · 1480 Saintes/Sint-Renelde**
- CH – IVF HARTMANN AG · 8212 Neuhausen**
- CZ – HARTMANN-RICO A.S. · 66471 Veverská Bítýška**
- DK – PAUL HARTMANN A/S · 3400 Hillerød**
- EG – PAUL HARTMANN Egypt S.A.E. · Giza, 12515**
- ES – Laboratorios UNITEX-HARTMANN S.A. · 08302 Mataró**
- FR – Lab. PAUL HARTMANN S.a.r.l. · 67607 Sélestat CEDEX**
- GB – PAUL HARTMANN Ltd. · Heywood/Lancashire OL10 2TT**
- GR – PAUL HARTMANN Hellas A.E. · 16675 Glyfada/Athina**
- HK – PAUL HARTMANN Asia-Pacific Ltd. · Hong Kong**
- HU – HARTMANN-RICO Hungária Kft. · 2051 Biatorbágy, Budapest**
- IT – PAUL HARTMANN S.p.A. · 37139 Verona**
- NL – PAUL HARTMANN B.V. · 6541 SH Nijmegen**
- NO – PAUL HARTMANN AS · 0377 Oslo**
- PL – PAUL HARTMANN Polska Sp. z o.o. · 95-200 Pabianice**
- PT – PAUL HARTMANN LDA · 2685-378 Prior Velho**
- RU – PAUL HARTMANN OOO · 115114 Moskwa**
- SE – HARTMANN-ScandiCare AB · 33421 Anderstorp**
- SG – PAUL HARTMANN Pte. Ltd. · Singapore 609 143**
- SI – PAUL HARTMANN Adriatic d.o.o. · 1000 Ljubljana**
- SK – HARTMANN-RICO S.R.O. · 81106 Bratislava**
- ZA – HARTMANN-Vitamed (Pty) Ltd. · Sandton 2128**

0 1 2 3