HARTMANN

# TenderWet® 24 active

**Aktiviertes Wundkissen zur Nasstherapie  
Activated wound dressing pad for wet therapy  
Pansement activé pour traitement en milieu humide**

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CE  
0 1 2 3

## Directions for use

TenderWet 24 active is a moist wound dressing whose central component is a superabsorbent polymer (SAP) activated by Ringer's Solution. The side facing away from the wound has a water-repellent layer, which ensures that moisture is released into the wound over 24 hours yet largely prevents the moisture from striking through the dressing.

## Properties and mode of action

TenderWet 24 active releases Ringer's Solution into the wound. Interactive and continuous wound irrigation takes place since the dressing absorbs wound secretions at the same time.

## Indications

For wet treatment of wounds with impaired healing.

## Mode of application

TenderWet 24 active is applied with the white side of the dressing on the wound so that the green markers face upwards. TenderWet 24 active must be in contact with the base of the wound and should overlap the wound edges slightly. Deep wounds must be plugged with TenderWet active cavity and covered with TenderWet 24 active. TenderWet 24 active should be secured with a bandage or, if necessary, with adhesive tape. TenderWet 24 active must be changed after 24 hours at most. If the wound requires a large quantity of moisture, the dressing can dry out prematurely. In such cases TenderWet 24 active should be moistened further between dressing changes. Ringer's Solution can be used to irrigate underneath the dressing before it is removed to ensure a gentle dressing change; it can then be removed painlessly after a few minutes.

## Side effects

At the start of the treatment with TenderWet 24 active, breakdown of irreversibly damaged tissue can produce an increase in the size of the wound. This is a sign that wound healing has commenced. Reddening of the wound edges can occur during wet therapy; this is usually a sign that the blood circulation has been reactivated as the wound begins to heal. There can be a tendency to spontaneous bleeding from the wounds during therapy with TenderWet 24 active. This is usually an indication that the wound is starting to heal. However, the cause must be clarified in each case. TenderWet 24 active therapy as such does not cause any pain. However, open wounds are often very sensitive to mechanical stimuli. It often helps if the dressing is moistened again after a few hours (without changing the dressing); otherwise temporary administration of appropriate analgesics is indicated.

## Contraindications

There are no known contraindications to date.

## Special precautions

TenderWet 24 active must not be cut to size or damaged mechanically in other ways. There is only limited experience of combining TenderWet 24 active with topical agents. Wet therapy with Ringer's Solution does not lead to maceration of vital cells. However, the plantar horny layer can swell, which is visible as a whitish layer. It is therefore advisable to apply a greasy ointment or a water-in-oil emulsion to the wound edges. It should be ensured that no residues remain on the skin.

CAUTION: Medical assessment of the wound condition and the causes of slow wound healing is necessary before treating wounds with impaired healing. Treatment with TenderWet 24 active cannot replace treatment of the cause of impaired healing.

## General instructions

Store below 25° C. Contents sterile unless peel pack is damaged. Do not re-sterilise. Keep out of reach of children.

AU – PAUL HARTMANN Pty. Ltd. · Homebush NSW 2140

GB – PAUL HARTMANN Ltd. · Heywood/Lancashire OL10 2TT

ZA – HARTMANN-Vitamed (Pty) Ltd. · Sandton 2128

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PAUL HARTMANN AG · 89522 Heidenheim, Germany
CH – IVF HARTMANN AG · 8212 Neuhausen